CervicalStim™

Spinal Fusion Therapy

CervicalStim Device Quick Instructions:

1. Press the **On/Off Button** to start.
2. The **LCD** will show your daily compliance percentage, then your prescribed daily treatment begins.
3. Wear the device over treatment site for **4 hours** every day.*
4. After treatment is complete, the device will power off.
5. To recharge, plug the charger into **Charging Port** and AC outlet.

*Clinical trial data demonstrates that success rates were significantly higher for patients who wore the device every day for the full treatment time. *Source: Premarket Approval P030034; FDA Approved December 2004.*

See the instruction manual for full usage information.

Orthofix products or services referenced herein are trademarks or registered trademarks of Orthofix Medical Inc. and its group of companies. All rights reserved.

Orthofix.com

BoneGrowthTherapy.com

P/N 20122284 Rev AD 2022-01-12
CS-2203 © Orthofix US LLC 01/2022

CervicalStim™

Spinal Fusion Therapy

CervicalStim Device Quick Instructions:

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Orthofix products or services referenced herein are trademarks or registered trademarks of Orthofix Medical Inc. and its group of companies. All rights reserved.

[Orthofix.com](https://www.orthofix.com)

[BoneGrowthTherapy.com](https://www.bonegrowththerapy.com)

P/N 20122284 Rev AD 2022-01-12
CS-2203 © Orthofix US LLC 01/2022

 **ORTHOFIX®**