

AccelStim™

Bone Healing Therapy



AccelStim Device Quick Instructions:

1. Connect the transducer to the AccelStim™ device by inserting the transducer cord into the transducer port.
2. Wear the strap such that the transducer will be on the area to be treated and secure it with the Velcro attached to the elastic strap.
3. Open the blue cover of the transducer holder by rotating it counterclockwise.
4. Apply the gel to the side of the transducer with no writing to form a 1-2 mm thick layer.
5. Insert the transducer inside the transducer holder so that the serial number is visible and then close the blue cover by rotating it.
6. Turn on the device by pressing On/Off Button for two seconds, until you hear a “beep”.
7. To start the treatment, press the Function Button of the device on the front panel.
8. Wear the device over treatment site for 20 minutes every day or as prescribed by your physician.
9. To recharge the battery, plug the power supply into the charging port.

AccelStim™

Bone Healing Therapy



Instrucciones rápidas del dispositivo AccelStim:

1. Conecte el transductor al dispositivo AccelStim™ insertando el cable del transductor en el puerto del transductor.
2. Ponga la correa de modo que el transductor quede sobre la zona que se va a tratar y sujétela con el Velcro unido a la correa elástica.
3. Abra la tapa azul del soporte del transductor girándolo en sentido contrario a las agujas del reloj.
4. Aplique el gel en el lado del transductor que no tiene escritura para formar una capa de 1 a 2 mm de espesor.
5. Introduzca el transductor en el soporte del transductor de modo que el número de serie quede visible y, luego, gire la cubierta azul para cerrarlo.
6. Encienda el dispositivo presionando el botón de encendido/apagado durante dos segundos, hasta que oiga un pitido.
7. Presione el botón de función del dispositivo en el panel frontal para iniciar el tratamiento.
8. Ponga el dispositivo sobre el zona de tratamiento durante 20 minutos todos los días o según lo prescrito por su médico.
9. Para recargar la batería, enchufe la fuente de alimentación en el puerto de carga.

Orthofix.com
BoneGrowthTherapy.com
1.800.535.4492

PN 20126754 REV AD 2022-11-29
AS-2103-US © Orthofix US LLC 11/2022

 **ORTHOFIX®**