

Quick Instructions

AccelStim™
Bone Healing Therapy



1. Turn the device on by pressing the Stand-by button and releasing it when you hear the beep. Then the display screen will light up.
2. Prior to treatment, the device will flash the text “GEL” to prompt the user to apply the ultrasound gel to the transducer (applied to the side with no writing).
3. Apply the gel to form a 1-2mm thick layer. Use a finger to spread the gel on the transducer to obtain an even layer and then close the blue cover by rotating it.
4. Click the arrow button on the bottom right side of the screen to acknowledge gel has been applied and continue to the treatment screen.
5. Wear the device over the treatment site for **20 minutes** every day or as prescribed by your physician.
6. To recharge the device, connect the power supply to the device. Plug the charger into the wall.

See the instruction manual for full usage information.

Orthofix, its logo, and AccelStim are trademarks or registered trademarks of Orthofix Medical Inc. and/or affiliate companies.



Orthofix.com
BoneGrowthTherapy.com



Instrucciones rápidas

AccelStim™
Terapia para consoli-
dación ósea



1. Encienda el dispositivo presionando el botón de espera y suéltelo cuando escuche el pitido. Luego se iluminará la pantalla.
2. Antes del tratamiento, el dispositivo parpadeará con el texto “GEL” para indicar al usuario que aplique el gel de ultrasonido en el transductor (aplicado al lado sin escritura).
3. Aplique el gel para formar una capa de 1-2 mm de espesor. Use un dedo para esparcir el gel sobre el transductor a fin de obtener una capa uniforme y luego gire la tapa azul para cerrarla.
4. Haga clic en el botón de flecha en la parte inferior derecha de la pantalla para confirmar que se ha aplicado gel y continuar con la pantalla de tratamiento.
5. Ponga el dispositivo sobre el zona de tratamiento durante **20 minutos** todos los días o según lo prescrito por su médico.
6. Para recargar el dispositivo, conecte la fuente de alimentación al dispositivo. Enchufe el cargador en la pared.

Consulte el manual de instrucciones para la información completa de uso.

Orthofix, su logotipo y AccelStim son marcas comerciales o marcas comerciales registradas de Orthofix Medical Inc. o compañías afiliadas.



Orthofix.com
BoneGrowthTherapy.com

