# Weighted Strap Accessory Guide

**AccelStim™**  
Bone Healing Therapy

Weighted Strap



Transducer Holder

Transducer

1. Place the transducer holder through the bottom opening of the weighted strap and open the blue cover to the transducer by rotating it.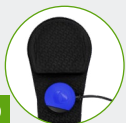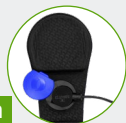

2. Insert the transducer into the transducer holder so the serial number is visible and facing upward. (As shown in picture 2a.) Close the blue cover by rotating it and proceed with treatment. (As shown in picture 2b.)

3. Apply the gel to the side of the transducer with no writing to form a 1-2mm thick layer. For more information please see the AccelStim device manual.

4. Place the assembled weighted strap over the area that needs to be treated. The transducer holder should align with the area of treatment. The transducer holder while attached to the weighted strap can be rotated as needed to provide maximum comfort.

5. At the end of the 20-minute treatment, remove the weighted strap assembly and clean the remaining gel from the weighted strap assembly. It is recommended to disassemble the weighted strap assembly to ensure the cleaning of remaining gel from each component. Refer to the Care and Cleaning instructions in the AccelStim device manual.

**Orthofix.com**  
**BoneGrowthTherapy.com**  
800.535.4492

PN 20127338 Rev AC 2025-08-18  
© Orthofix US LLC 08/2025

**Warning:** If any parts of the AccelStim device or accessories are damaged, do not use the AccelStim device. Please contact Patient Care at 800.535.4492.

**Precaution:** Patients and caretakers should assess for adverse reactions on the skin where the device has contact, such as redness (erythema), swelling (edema), irritation, sensitization (delayed Type IV hypersensitivity), allergy, immune response, or other reactions. Stop use if any reactions are detected.

Orthofix, its logo, and AccelStim are trademarks or registered trademarks of Orthofix Medical Inc. and/or its affiliate companies.

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